



RespiPatient® Quick Setup Guide





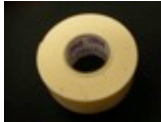
RespiPatient Components

Item	Picture
RespiPatient Manikin in black zippered carrying case	
RespiPatient Interface Box	
Cords	
RespiPatient Power Cord	
ASL 5000 to RespiPatient Interface Box 6" Power Cord	




RespiPatient® Quick Setup Guide

Item	Picture
RespiPatient to ASL 5000 36" Dual Plug 14 Pin Gray Cord	
Tubing	
Left & Right Lung Inflation Tubing	
Pneumothorax Tube	

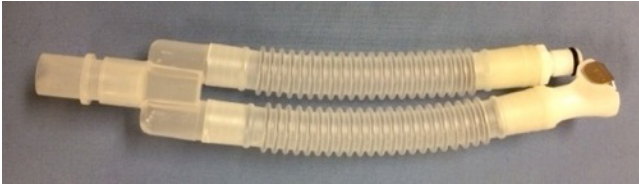RespiPatient® Quick Setup Guide

Item	Picture
Tongue Inflation Tube	
Trachea-to-ASL 5000 Circuit	
Bypass Tubing	
Consumable Items	
1 bottle of TL001 lubrication	
1 roll of 3M microfoam tape	

RespiPatient® Quick Setup Guide

Item	Picture
2 packages of neck tube patches	
2 chest tube drainage patches: one left and one right.	
2 neck covers	
Additional Items	
RespiPatient Intubation Package	
RespiPatient Decompression Needle	
Drape	

RespiPatient® Quick Setup Guide

Connecting RespiPatient, the RespiPatient Interface Box, and the ASL 5000 Breathing Simulator

1) Getting Started:


Place the RespiPatient, the RespiPatient Interface Box and the ASL 5000 on a table in close proximity to each other. Stack the ASL 5000 on top of the RespiPatient Interface box by placing the ASL 5000 feet into the orange cups on top of the Interface box.

2) Making Connections:


RespiPatient Manikin to RespiPatient Interface Box

Lungs:

Left Lung Connection: Insert the female end of a lung tube into the Interface box Lung port labeled "L." Insert the male end of the same lung tube into the RespiPatient Left Lung port.

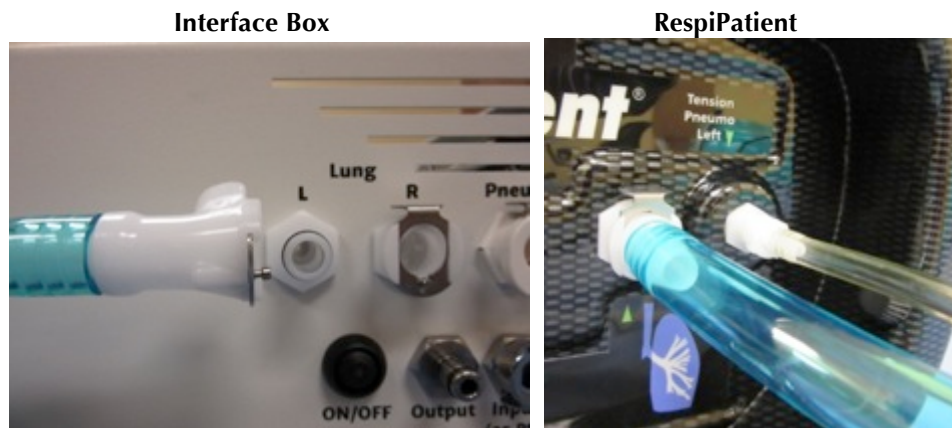

Right Lung Connection: Insert the male end of a lung tube into the Interface box Lung port labeled "R." Insert the female end of the same lung tube into the RespiPatient Right Lung port.

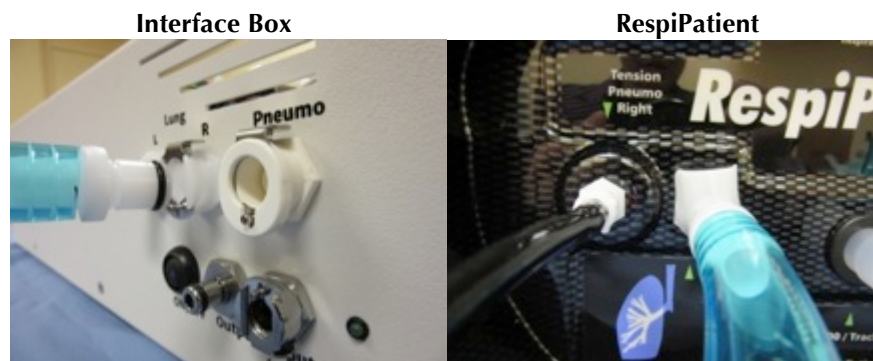

RespiPatient® Quick Setup Guide

Both Lung Tubes plugged into the Interface Box



Pneumothorax:

Interface Box Connection: Insert the single-connector end of the Pneumothorax tubing into the Interface box port labeled “Pneumo.”



RespiPatient® Quick Setup Guide

RespiPatient Connection:

Insert the other ends of the Pneumothorax tubing into the RespiPatient as follows:

Insert the **clear** tubing into the port labeled “Tension Pneumo Left.” Then turn the connector slightly to the right (Clockwise) to snap it into place.



Insert the **black** tubing into the port labeled “Tension Pneumo Right.” Then turn the connector slightly to the right (Clockwise) to snap it into place.



RespiPatient® Quick Setup Guide

RespiPatient Manikin to ASL 5000

Trachea:

RespiPatient Connection: Insert the elbow end of the trachea tube into the port labeled “ASL 5000/Trachea” as shown.



ASL 5000 Connection: Insert the other end of the trachea tube into the front panel of the ASL 5000 as shown.



RespiPatient® Quick Setup Guide

RespiPatient Interface Box to ASL 5000

Short Power Cord Connection:

Connect one end of the 6" Power Cord into RespiPatient Interface Box power socket as shown and the other end into the ASL 5000 power socket as shown.

Interface Box



ASL 5000



Interconnect Cable:

Connect one end of the interconnect cable into the back of the RespiPatient Interface Box and the other end of the cable into the back of the ASL 5000 as shown.

Interface Box



ASL 5000




RespiPatient® Quick Setup Guide

RespiPatient Interface Box

Power:

Connect the power cord to the RespiPatient Interface box as shown. Insert the other end into an AC power source.

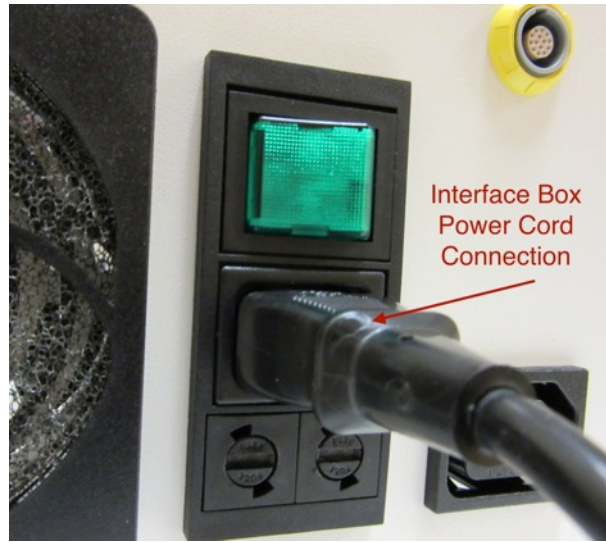

Final Manikin Connection

Tongue Inflation Tubing:

Insert the connector end of the tongue inflation tube into the manikin in the black connector labeled “Tongue Inflation” as shown. Then turn the connector slightly to the right (Clockwise) to snap it into place.



RespiPatient® Quick Setup Guide

Bypass Mode

To operate the RespiPatient in stand-alone mode without the ASL 5000, connect the Bypass Tubing as shown in the photo below.

